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Parallelepiped-shaped strips of rapid-resorption expanded collagen with a haemostatic function for rapidly filling large residual bone cavities and allowing guided bone and periodontal tissue regeneration.

“System of guided regeneration materials  
Collagene AT<sup>®</sup> Idrossilapatite AT<sup>®</sup>”

### General particulars and technical characteristics

Rapid-resorption **Collagene AT<sup>®</sup> Espanso 6x6x60 mm** is indicated in all those cases in which it is necessary to rapidly fill large residual bone cavities following extractions or the removal of dental cysts. It has an efficacious haemostatic effect inside the cavities and sockets and allows hydroxylapatite and Collagene AT membranes to adhere to the outer surface of the parallelepiped of expanded collagen, so that substances can be introduced into the residual bone cavities and post-extraction sockets to achieve guided bone regeneration.

**Collagene AT<sup>®</sup> Espanso 6x6x60 mm** was studied and is obtained using lyophilised collagen of equine origin.

**Collagene AT<sup>®</sup> Espanso 6x6x60 mm** has a microrough surface and the microspaces with a diameter of 10-20µ between the layers of cross-linked collagen form an efficacious barrier that prevents cell migration.

### Packaging and storage

Each 6x6x60 mm parallelepiped of **Collagene AT<sup>®</sup> Espanso** is packed in a vial with an airtight screw-cap.

Each pack of **Collagene AT<sup>®</sup> Espanso** contains 5 vials.

**Collagene AT<sup>®</sup> Espanso 6x6x60 mm** is gamma ray-sterilised, disposable and must be stored below 40°C.

*Do not autoclave.*

### Handling

All instruments used for handling must be sterile.

Open the vial with airtight screw-cap and, using dental forceps, carefully remove the strip of rapid-resorption Collagene AT<sup>®</sup> Espanso.

For **use in post-extraction haemostasis**, carefully position one or more strips of Collagene AT<sup>®</sup>

**Espanso** over the post-extraction socket and press the strips down until they completely fill the post-extraction socket, so that no further compression is possible.

For **guided bone regeneration in post-extraction sockets and bone cavities**, take one or more strips of Collagene AT<sup>®</sup> Espanso; once Idrossilapatite AT<sup>®</sup> wetted with sterile saline solution has been made to adhere to the outer surface, using one or two dental forceps, carefully position them over the post-extraction socket and press down until the cavity is completely filled, so that no further compression is possible.

The socket or surgical wound must be protected with surgical packing until it has healed.

**Collagene AT<sup>®</sup> Espanso 6x6x60 mm** is indicated for the filling of post-extraction sockets, periodontal intraosseous pockets and sinus lifts, for atrophic bone crest augmentation, in periapical surgery after apicectomies and in all other oral surgery settings requiring bone regeneration

### Warnings

**Collagene AT<sup>®</sup> Espanso** must not be used in the presence of acute infection, infected lesions or intolerance to tissues of equine origin.

The medical device **Collagene AT<sup>®</sup> Espanso** is intended for use by Dental surgeons and specialist doctors only.

Do not use the medical device unless the packaging is intact.

The adhesive labels provided for traceability should be applied to the patient's records.

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